

**ZOLL AUTOMATED EXTERNAL  
DEFIBRILLATORS (AED) PLUS CHECKLIST**

Daily inspection to verify the green checkmark is present indicating unit is ready for use. If green checkmark present AED unit passes. If green checkmark is not present AED unit fails and contact Biomedical Repair. Write initials in the appropriate column.

Month/Location:	Week of:									
	PASS	FAIL								
Monday										
Tuesday										
Wednesday										
Thursday										
Friday										
Saturday (24/7 manned spaces only)										
Sunday (24/7 manned spaces only)										

MONTHLY INSPECTION DATE: _____	PASS	FAIL	COMMENTS
1. Turn on unit and perform Manual Self Test			
2. Turn off unit, ensure the green checkmark is present indicating ready for use.			
3. Electrodes are within their expiration date and sealed in package			
4. Electrodes are pre-connected to input connector			
5. Supplies package intact and available for use			
6. The unit is clean, undamaged and free of excessive wear			
7. All cables are free of cracks, cuts and exposed or broken wires			

**CLEANING THE UNIT:**

- After each use clean and disinfect the unit with a soft damp cloth using 90% isopropyl alcohol, soap and water or chlorine bleach/water mixture (30mL/liter water)
- Do not immerse any part of the unit in water
- Do not use ketones ( MEK, acetone, ect) to clean the unit
- Avoid using abrasive (e.g. paper towel) on the display window or IrDA port
- Do not sterilize the device.