

Philips® Heartstart FRx Automated External Defibrillator (AED)

**DAILY CHECKLIST**

Month: \_\_\_\_\_ Year: \_\_\_\_\_ Location: \_\_\_\_\_ Unit Serial No.: \_\_\_\_\_

Instructions	Recommended Corrective Action
1. Examine the wall unit and glass cover for any foreign substances, damages, or cracks.	Remove item and/or clean the container.
2. Verify a slow consistent green blinking light on the AED unit.	If light is not blinking, contact Bio-med repair at ext 4-6960.
3. Are medical gloves present inside the container?	If not present, notify CAPO / ACAPO.
4. Is there a pocket mask present inside the container?	If not present, notify CAPO / ACAPO.

If all checks are correct initial and date below:

	SUN	MON	TUES	WEDS	THURS	FRI	SAT
Date/Initials	/	/	/	/	/	/	/
Date/Initials	/	/	/	/	/	/	/
Date/Initials	/	/	/	/	/	/	/
Date/Initials	/	/	/	/	/	/	/
Date/Initials	/	/	/	/	/	/	/
Date/Initials	/	/	/	/	/	/	/

If corrective action needed, document below:

Date	Discrepancies	Action Taken	Date Resolved

Initials	Printed Name/Rank	Initials	Printed Name/Rank